

**Collector's Quiz: OraSure® HIV-1 Collection**  
**Revised March 2002**

Attention Insurance Agents: Print this document, complete all 14 questions of the examination, and mail or fax your printed page with your answers clearly indicated to your company's underwriting department.

Name:	
Company:	
Department:	
Contact Information:	

**OraSure® HIV-1 Collection Quiz**

1. The OraSure HIV-1 Oral Specimen Collection Device is designated to collect:
  - a. Oral fluid
  - b. Mucus
  - c. Blood
  - d. All of the above
  
2. The Collection Pad should be left in the subject's mouth for:
  - a. At least five minutes, but no more than eight minutes
  - b. At least three minutes, but no more than ten minutes
  - c. At least two minutes, but no more than five minutes
  - d. Exactly one and a half minutes
  
3. The Collection Pad should be placed:
  - a. In one of several body openings
  - b. In the mouth, between the upper cheek and gum
  - c. Under the tongue
  - d. In the mouth, between the lower cheek and gum
  
4. Before specimen collection, subjects should:
  - a. Be encouraged to ask questions
  - b. Be advised of the limitations of the procedure, told about the clinical data, and advised of other available options for HIV testing
  - c. Be give the "Subject Information" pamphlet, containing information on OraSure, HIV and AIDS
  - d. All of the above

5. When the Collection Pad is removed from the pouch:
  - a. It should be grasped by the plastic handle
  - b. It should be grasped by the pad
  - c. The plastic handle should be snapped off
  
6. After specimen collection, the Collection Pad is placed:
  - a. Into the Specimen Vial by forceps
  - b. Into the Specimen Vial by the subject
  - c. Into the Specimen Vial by the collector
  - d. Into a sealable plastic bag
  
7. A subject whose OraSure HIV-1 specimen tests positive for the first time should:
  - a. Be advised to have another OraSure HIV-1 specimen taken and tested
  - b. Be advised to have a blood specimen tested
  - c. Be advised to ignore the result
  - d. Either a or b are correct
  
8. A subject should be counseled that test results are wrong:
  - a. More often when testing an OraSure HIV-1 specimen than a blood specimen
  - b. Less often when testing and OraSure HIV-1 specimen than a blood specimen
  - c. The same number of times when an OraSure HIV-1 specimen or a blood specimen is tested
  - d. None of the time
  
9. After specimen collection, the nylon stick is broken by:
  - a. No one, it remains intact
  - b. A quick snap by the test subject
  - c. Forceps
  - d. The collector
  
10. The Specimen Vial should:
  - a. Have the subject identification information recorded on it
  - b. Have the date of collection recorded on it
  - c. Be protected from breaking
  - d. All of the above
  
11. OraSure HIV-1 specimens collected in a non-medical setting:
  - a. Do not require any special packaging
  - b. Should be protected from temperature over 98° F
  - c. Are inappropriate because the device can only be used in a doctor's office
  
12. False Negative and False Indeterminate results:
  - a. Occur more often when testing with OraSure HIV-1 specimens than with blood specimens

- b. Occur less often when testing with OraSure HIV-1 specimens than with blood specimens
  - c. Never happen
13. HIV-1 test results with OraSure HIV-1 specimens rely upon:
- a. Two or three different lab tests
  - b. A screening test and a supplemental confirmatory test
  - c. A screening test only
  - d. A Western Blot test only
14. As a collector, I should be prepared to answer questions about:
- a. Preventing AIDS
  - b. The OraSure HIV-1 Oral Specimen Collection Device
  - c. The meaning of a positive, a negative and an indeterminate test result
  - d. All of the above